Instructions for Use

OXYGUARD®

Disposable Endoscope Biteblock with Oxygen Access

Oxyguard D and Oxyguard Mini

WARNING: The Oxyguard is for single patient use. To avoid the risk of cross-infection, do not reuse between patients.

CAUTIONS:

- ☒ Care should be exercised when administering supplemental oxygen to patients with chronic lung disease.
- ☒ It is recommended that arterial oxygenation is monitored when administering supplemental oxygen.
- ☒ US federal law restricts sale of this device to, or on the order of, a physician.



INTENDED USE: The Oxyguard is a biteblock intended to protect the patient and instrument during upper-gastrointestinal endoscopy. It includes an oxygen port to allow supplemental oxygen to be delivered simultaneously to the nose and mouth.

The Oxyguard is intended for use by medical staff competent in the management and maintenance of a clear airway in an unconscious or sedated person. It is intended for adult patients.

The Oxyguard Mini is a smaller biteblock for individuals with restricted opening of the mouth and can also be used for children.

INSTRUCTIONS FOR USE

1. Remove the biteblock from the packaging.
2. If required, attach a head strap (available separately) to the notches on either side of the biteblock.
Note: Use of a head strap is optional. It may help keep the biteblock in position during endoscopy.
3. If supplemental oxygen is required, attach the oxygen line to the port on the side of the biteblock (see Figure 1).
4. Position the Oxyguard in the patient's mouth (see Figure 2), and fit and adjust the head strap (if one is used).
5. Adjust the oxygen flow accordingly.
Note: Oxygen will flow to the patient's nose and mouth.
6. If a head strap was used, detach this from the bite block first before removing the biteblock from the patient's mouth.
7. After use, dispose of product appropriately as a contaminated biohazard.

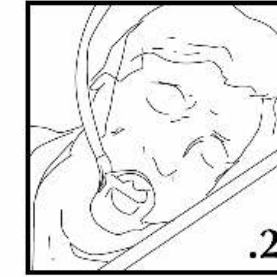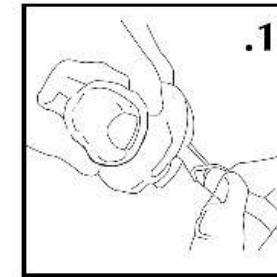Trawax Pty Limited
28 Queenscliff Road
Queenscliff
NSW 2096, Australia
www.trawax.com.au

Distributed in Australia by:
Vertrieben in Australien durch/Distribué en Australie par :
Endomed Pty Ltd
Ph: +61 7 3881 1883
Fx: +61 7 3881 0211

Advena Ltd.
Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

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©Trawax Pty Ltd 2007. Rev 09 11/19
Oxyguard is a registered trademark of Trawax Pty Ltd.